

GC Ortho Connect™ Flow

Light Cure Orthodontic Adhesive
One Step System



Since 1921
Towards Century of Health


Experts in Orthodontics
GC Orthodontics Europe GmbH
www.gorthodontics.eu

GC Ortho **Connect™** Flow 90-3100-0000 (2 syringes of 2,7 g each)

Light Cure Orthodontic Adhesive for Lingual Retainers, Stops and Aligner Attachments

Easy to Handle

Like GC Ortho Connect™, GC Ortho Connect™ Flow's primer is integrated into the paste, allowing to save valuable time in the bonding process. GC Ortho Connect™ Flow makes it especially easy to bond lingual retainer wires.

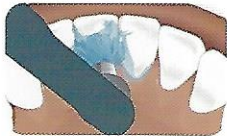

Clean and dry




Etch





Position retainer



Apply Ortho Connect Flow

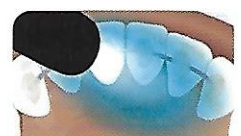

Light Cure

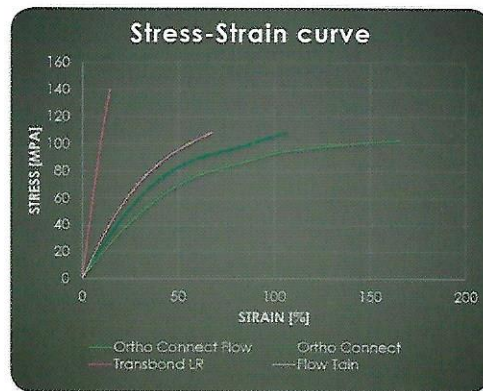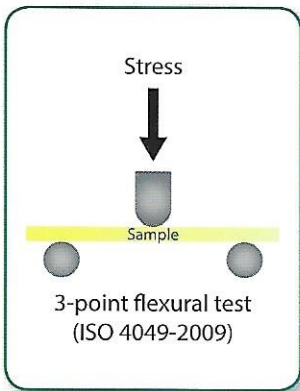


Improved Flexibility and Resistance

Thanks to a new long chain monomer, the adhesives flexibility is enhanced.

Improved Flexibility: Increased shock resistance and resin strength

High Toughness: The resin is hard to break

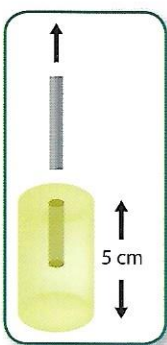


GC Ortho Connect Flow

GC Ortho Connect

Competitor 1

Competitor 2

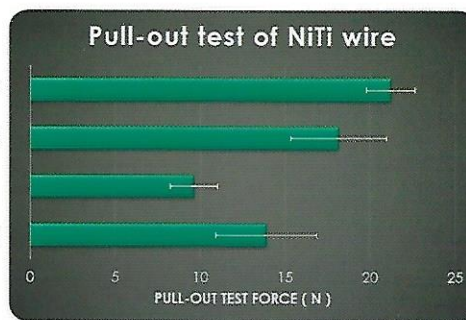

Strong adhesion to retainer wires or archwires

Ortho Connect™ Flow is auto-adherent and bonds extremely well to both enamel and metal. Thanks to its phosphate monomer, similar to a metal primer, Ortho Connect™ Flow ensures a strong bond on wires.

GC Orthodontics Europe GmbH
Harkortstrasse 2
D-58339 Breckerfeld
Germany

